

## **Step One: Registration**

### **From an on-campus computer:**

- Go to <http://nursing.unboundmedicine.com>
- Select 'Mobile' from menu in top right hand corner
- Click 'Register Now'
- Accept the terms and conditions
- Fill in and submit the online registration form. Please remember your username and password. This will be used at a later date. -A Nursing Central registration email will also be sent with your serial number.
- You will then be brought to a 'Registration Complete screen'. You are done for now.

## **Step Two: Installation**

### **Palm OS and Windows Mobile (Pocket PC) Installation Instructions:**

#### *On a computer:*

- (1) Point your browser to: <http://www.unboundmedicine.com/support/ub?>
- (2) Under 'Download Software', enter your serial number and email address, and click download
- (3) Follow on-screen instructions

### **BlackBerry Installation Instructions:**

\*A word of caution – this is a big download! Before you begin this process please attach the charger to your device and plug the charger into the wall. The download will take approximately 40 minutes.

#### *On the BlackBerry:*

- (1) Open up your BlackBerry browser and go to <http://www.ubmed.com/bb/409>
- (2) Read the instructions on the website and click the appropriate link (touch screen or non-touch screen).
- (3) Follow the on screen instructions to complete the registration and download process.
- (4) Once the software has downloaded to your Blackberry, press 'Run' to open immediately or 'OK' to open later. If you open later, the program can be found in the download folder.
- (5) Enter your serial number to unlock the content

### **iPod Touch / iPhone Installation Instructions:**

#### *On the iPhone/iPod Touch (with wireless internet):*

- (1) Go to the App store on your device.
- (2) Search for "Unbound" and tap **Unbound Medicine, Inc.**
- (3) Select **Nursing Central**, tap Free, then tap Install
- (4) You will be asked to login to the App Store (using your iTunes password)
- (5) Nursing Central will then install to your device.
- (6) Once installed, tap the Nursing Central icon and login with your username and password.

### **Android Installation Instructions :**

\*A word of caution – this is a big download! Before you begin this process please attach the charger to your device and plug the charger into the wall. The download will take approximately 40 minutes.

#### *On the smartphone:*

- (1) Open up your Android browser and go to <http://www.ubmed.com/an/419>
- (2) Read the instructions on the website and click the appropriate link (touch screen or non-touch screen).

- (3) Follow the on screen instructions to complete the registration and download process.
- (4) Once the software has downloaded to your Android smartphone, press 'Open' to open immediately or 'Close' to open later. If you open later, the program can be found on the main menu screen.
- (5) Enter your serial number to unlock the content

**Technical Support Information:**

A list of frequently asked questions can be found at <http://www.unboundmedicine.com/support/ub>

Any questions or concerns can be directed to our complimentary in-house support staff. Support can be reached at [support@unboundmedicine.com](mailto:support@unboundmedicine.com) or by dialing 610-627-9090 and selecting phone option 1.